


TOSI®-FlexiCheck Instructions for use

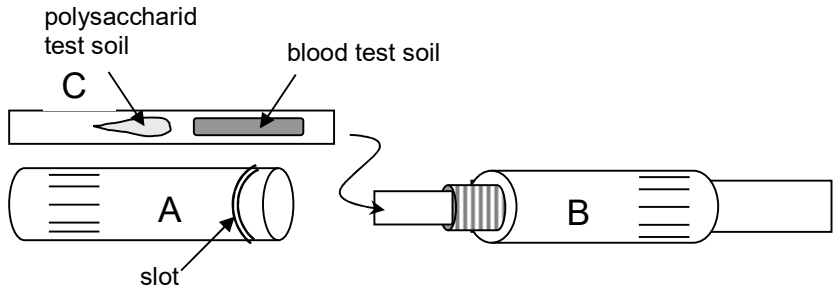

1. Unscrew FlexiCheck-device: detach part A (marked with slot) from part B (see picture).
2. Open protective pouch of TOSI®-FlexiCheck and insert the test object (part C) in part B as shown in picture. Do not touch the area covered with test soil.
3. Close FlexiCheck-device again.
4. Connect FlexiCheck-device with one of the channel irrigation system of the washer (luer-lock) and start the cycle according to your manufacturer manual instructions.
5. Open the device after the reprocessing cycle: disconnect part A from part B to remove the TOSI®-FlexiCheck without touching the test soil area!
6. Visual inspect the result of the TOSI®-FlexiCheck.

STORAGE:

Store in protective pouch at 2°C to 25°C. Keep away from light, heat and chemicals.

CONTENT: 25 tests

Do not open TOSI®-FlexiCheck protective pouch before use

Shelf life: see imprint

Marketing & Sales:

Manufacturer:

PEREG GMBH
Porschestr. 12
D-84478 Waldkraiburg
Germany